

VITASEPT[®]WIPES med

ALCOHOL-BASED DISINFECTING WIPES

ready-to-use

VITASEPT[®]WIPES med is a ready-to-use disinfectant for rapid disinfection, suitable for use on alcohol resistant surfaces of medical devices. Doesn't leave residues on surfaces, thus it is suitable also for the disinfection of surfaces that come in contact with food and food products.

Conforms to requirements in healthcare, food industry, beauty services etc.

PRODUCT PROPERTIES

- fast exposure time
- alcohol-based
- doesn't contain aldehydes, quaternary ammonium compounds and chlorine ions
- doesn't leave residues
- particularly suitable for hard-to-reach places
- doesn't need to be washed off
- doesn't change the visual appearance of the surface

USAGE

Pull the wipes out until perforated spot according to the amount necessary and if needed support with other hand in order to tear away. All surfaces should be wiped with Vitasept[®]Wipes med, make sure that the surface is covered with disinfectant. Throw the used wipe away. In order to avoid the drying of wipes, close the flip.

EXPOSURE TIME

60 seconds

MICROBIOLOGICAL SPECTRUM

Bactericidal EN 13727	60 seconds
Bactericidal and fungicidal EN 13697	60 seconds
Bactericidal and fungicidal (wipe method) EN 16615	60 seconds
Fungicidal EN 13624	60 seconds
Tuberculocidal EN 14348	30 seconds
Mycobactericidal EN 14348	60 seconds
Full virucidal EN 14476, EN 16777	60 seconds

CONFORMITY TO STANDARDS

EN 13624, EN 13697, EN 13727, EN 14348, EN 14562, EN 14563, EN 14476, EN 16777, EN 16615, EN 17387, DVV/RKI

STORAGE

See the best before date on the package. Store at temperatures 5-25°C and keep away from direct sunlight to assure the quality of the product.

UTILIZATION

Product is ecologically safe, all active ingredients are biodegradable in flush water. Plastic packaging materials are recyclable. For collection, the plastic packaging is suitable when rinsed, the bottle and cap separated. Spray with its metal parts is also suitable for plastic collection. The packaging box is sorted for cardboard collection. The compatibility of the packaging and the product is quality assured.

SAFETY INFORMATION

See the product safety data sheet for additional information. If in relation to the use of VITASEPT®WIPEs a serious deterioration of health occurs, it should be reported to the manufacturer and the competent authority of your country. When in doubt, consult with an advisor on regulatory affairs.

COMPOSITION

100g of the product contains active ingredients: propan-2-ol 60%, ethanol 15%

pH~7

Product bears the CE marking according to Medical Device directive 93/42/EEC
Biocide reg.no: 2255/21



PACKAGES: 24wipes, 120wipes, 150wipes, 200wipes, 600wipes



Goodpoint Chemicals OÜ
Urda tee 2/1, Jälgimäe 76404,
Saku vald, Harjumaa, Estonia
www.goodpointchemicals.com
info@goodpointchemicals.com

