

VITASEPT®P WIPES

ALCOHOL-BASED WIPES

VITASEPT®P WIPES is a ready-to-use disinfectant for non-invasive medical devices and alcohol-resistant surfaces.

Product has bactericidal, mycobactericidal, virucidal and fungicidal efficacy.

Solution doesn't leave residues on surfaces; thus, it is suitable also for the disinfection of surfaces that come in contact with food and food products.

This product is considered as dual use product and conforms requirements according to Regulation (EU) 2017/745 on medical devices and Biocidal Products Regulation (BPR, Regulation (EU) 528/2012) PT2, PT4.

PRODUCT PROPERTIES

- fast exposure time
- ethanol-based
- doesn't contain aldehydes, quaternary ammonium compounds and chlorine ions
- doesn't leave residues
- particularly suitable for hard-to-reach places
- soft fabric-like material, doesn't contain cellulose and doesn't leave lint on surfaces
- doesn't need to be washed off
- doesn't change the visual appearance of the surface

USAGE

Pull the wipes out until perforated spot according to the amount necessary and if needed support with other hand in order to tear away. All surfaces should be wiped with VITASEPT®P WIPES, make sure that the surface is covered with disinfectant. Throw the used wipe away. In order to avoid the drying of wipes, close the flip.

EXPOSURE TIME

60 seconds

MICROBIOLOGICAL SPECTRUM

Bactericidal ¹⁾ EN 13727	30 seconds
Bactericidal ¹⁾²⁾ and yeasticidal ¹⁾²⁾ EN 16615	60 seconds
Yeasticidal ¹⁾ EN 13624	30 seconds
Fungicidal ¹⁾ EN 13624	60 seconds
Mycobactericidal ¹⁾ EN 14348	30 seconds
Virucidal ¹⁾ EN 14476	60 seconds

1)clean condition 2) dirty condition

CONFORMITY TO STANDARDS

EN 13624, EN 13727, EN 14348, EN 14476, EN 16615

STORAGE

See the best before date on the package. Store at temperatures 5-25°C and keep away from direct sunlight to assure the quality of the product.

UTILIZATION

Plastic packaging materials are recyclable. For recycling, rinse package with water. The packaging box is sorted for cardboard collection. The compatibility of the packaging and the product is quality assured.

SAFETY INFORMATION

See the product safety data sheet for additional safety information.

COMPOSITION:

Active ingredients of the solution: ethanol 80% (g/g)

pH ~7

Product bears the CE marking according to Regulation (EU) 2017/745 on medical devices

Product corresponds to Biocidal Products Regulation (BPR, Regulation (EU) 528/2012) PT2, PT4

Biocide reg.no: 2212/21

PACKAGES: 24wipes flow bag, 120wipes flow bag, 150wipes jar, 200wipes flow bag/jar, 600wipes jar



Goodpoint Chemicals OÜ
Urda tee 2/1, Jälgimäe 76404,
Saku vald, Harjumaa, Estonia
www.goodpointchemicals.com
info@goodpointchemicals.com

