



# Globasept®Wound Sol.

## WOUND IRRIGATION SOLUTION

Globasept®Wound Sol. is suitable for cleaning, rinsing, moisturizing acute and chronic wounds (incl. scratches, abrasions, bites, tears etc). Accelerates healing due to wound cleanliness. Wounds need enough moisture for optimal healing, as the skin cells must be able to divide and move for the healing process to take place. This is best possible in a humid wound environment.

---

### PRODUCT PROPERTIES

- colourless
- doesn't sting or irritate
- doesn't stain
- suitable for both first aid and wound care
- also suitable for use in children
- also suitable for use in animals (prefer flip-cap bottle)

### APPLICATIONS:

- cleaning and antiseptics of superficial wounds (abrasions, etc.)
- cleaning and antiseptics of bite and claw mark wounds
- cleaning and antiseptics of cut and tear wounds
- care of chronic wounds (eg bedsores, diabetic ulcers)
- surgical wound care
- cleaning and antiseptics of burns (including sunburn)
- post-piercing wound care and antiseptics
- relief for insect bites
- oral antiseptics (in dental)

---

### USAGE

Irrigate the required area evenly and completely.

#### **Spray method (eg for wounds):**

Spray the product abundantly over the entire area, making sure that the surface is evenly moistened.

#### **Cleaning with a gauze:**

Thoroughly moisten a sufficient amount of sterile gauzes and clean the required area completely.

#### **Insect bites and sunburn:**

Thoroughly moisten a sufficient amount of gauzes and clean the required area.

---

### STORAGE

See the best before date on the package. Store at temperatures 5-25°C, keep away from direct sunlight.

---

## UTILIZATION

Product is ecologically safe, all active ingredients are biodegradable in flush water. Plastic packaging materials are recyclable. For collection, the plastic packaging is suitable when rinsed, the bottle and cap separated. The pump and spray with its metal parts are also suitable for plastic collection. The packaging box is sorted for cardboard collection. The compatibility of the packaging and the product is quality assured.

---

## WARNINGS!

The product is for external use only, not suitable for infusion or injection! Do not swallow! In very rare cases, a slight burning sensation may occur, but this dissipates after a few minutes. Globasept®Wound Sol. should not be used if you are allergic to any of the ingredients in this product. The product is not suitable for use in the middle or inner ear, hyaline cartilage or in the eyes. Avoid inhalation of aerosol. Do not use in combination with anionic surfactants as they may interact with polyhexanide. Due to insufficient clinical experience with pregnant and breastfeeding women, newborns and infants, the product should only be used after careful medical evaluation. Do not use a product with damaged packaging. Bottles in contact with the wound must be discarded after use. If the wound does not begin to heal within a few days or if swelling, redness or pain develops in the wound area, consult a physician. Do not use consistently for longer than 1 month. Medical device - Keep out of reach of children.

---

## COMPOSITION

100g of the product contains: polyaminopropyl biguanide (polyhexanide) 0,1%

Product bears the CE marking according to Medical Device directive 93/42/EEC



**PACKAGES:** 100ml flip-cap, 100ml spray, 400ml flip-cap

---



Goodpoint Chemicals OÜ  
Urda tee 2/1, Jälgimäe 76404,  
Saku vald, Harjumaa, Estonia  
[www.goodpointchemicals.com](http://www.goodpointchemicals.com)  
[info@goodpointchemicals.com](mailto:info@goodpointchemicals.com)

